# Instruction for Use

# EYE GENIC

(Soft Contact Lens)

Japan Approval No. 22300BZX00162A45

MDA Reg. No. GB1645123-138738

*Please read carefully and keep this information for your use.*

## Contents

### 1. Composition of the lens

- |   |  |
|---|--|
| 1) Grouping of soft contact lenses :                              | Group I  |
| 2) Constituent monomer :  | 2-HEMA, EGDMA  |
| 3) Water content :  | 38.5%  |
| 4) Oxygen permeability coefficient                                | $10.5 \times 10^{-11}$ (cm <sup>2</sup> / sec), (mLO <sub>2</sub> / (mL x mmHg))     |
| 5) Coloring agent :   | Metal Oxide Colorants,<br>Quinoline Coloring Agent,<br>Anthraquinone Coloring Agent. |
| Place of the colored surface &<br>Condition of the colored part : | Colorant is embedded in the lens by a polymer layer.                                 |

### 2. Preservation Fluid

Main ingredients of preservation solution : Sodium Chloride

### 3. Principle

- In the case of reusable vision-correcting colored contact lenses, a colored design is crafted into the lens to change the appearance of the iris or pupil when worn on the eye by the user. In addition, the lens has a degree, which has a vision correction effect.

## Indication for Use (Intended Purpose)

### [ Purpose of use or effect ]

- In the case of reusable vision-correcting colored contact lenses, vision correction and changing the appearance of the iris or pupil when worn.
- In the case of reusable non-vision correcting colored contact lenses, changing the appearance of the iris or pupil when worn.

## How to Use

All-day use, Replace up to one month, Scientific disinfection.

### 1. Lens on and off

- 1) Precautions for handling the lens
  - Cut your nails short and smooth then into a round shape.
  - Always clean your hands before handling the lens.
  - When putting on and taking off the lens, avoid sticking your fingernails or putting your fingertips in direct contact with your eyes.
- 2) How to take out the lens
  - Be careful not to cut your fingers with the aluminum seal when you open it.
  - Be gently remove the lens from the container by your fingertip, without give damage on the lens by your fingernail.
  - Check the left/right and front/back sides of the lens before use.
- 3) How to wear lens on
  1. Place the lens on your fingertip of index finger .
  2. While look in the mirror, open your eyes wide with the thumb and index finger of your other hand.
  3. Slowly bring the lens closer to the eye and gently wear lens it over the black eye.
  4. Once you have wear lens on your black eye, slowly release the finger that was opening your eye and blink slowly your eyes.
  5. Make sure check that wear lens was correctly by left and right view.

- 4) How to remove lens
  1. Pull down the lower eyelid with your middle finger.
  2. Pinch the bottom of the lens with your thumb and index finger and remove it.

## 2. Applying Schedule

- 1) This lens is an all-day wear lens that should be replaced in up to a month. The lenses should be worn while you are awake and removed before you go to bed.
- 2) The schedule for getting used to wearing the lenses varies by personal. Be follow instructions by eye doctor and familiarize yourself with the schedule that use best for you.

< example >

Day 1 - 3	3 hours
Day 4 - 6	6 hours
Day 7 ~	a full-day

- 3) If you interrupt the use of the lens ;
  - For less than a week, you can use the lens as usual.
  - If you interrupt for more than a week, start with the example schedule as if you were using the lens for the first time.
  - If interrupted for more than one month, consult an eye doctor before starting the application.

## 3. Lens Care

Must be essential to clean and disinfect soft contact lenses after wearing them. As a reminder, please observe the followings.

- 1) Thoroughly scrub both sides of the lens.
- 2) Rinse before wearing the lens.
- 3) The MPS should be used approximately one month after opening the package.
- 4) Do not reuse the disinfectant after use.
- 5) Do not replace the disinfectant with another container.

For more information, be sure to read the instructions and indications for each care product.

## 4. How to manage the lens case

- 1) The lens case should be replaced periodically with a new one.
- 2) After use, discard the disinfectant inside the lens case, wash it thoroughly, and let it dry naturally.

## 5. Regular Checkup

Be sure to have regular check-ups one week, one month, three months, and every three months thereafter, or as directed by your eye doctor.

# Precautions for Use

## 1. Important Basic Notes

- 1) Always read this instruction manual carefully before wearing the lens and keep it ready to read when you need.
- 2) If you have an allergic disease, consult an eye doctor as the risk of adverse events increases.

## 2. Trouble, Adverse event

The following defects or adverse events may occur.

### 1) Trouble

Lens : Torn, scratched, deformed, discolored, foreign matter attached.

Solution & package : Leakage, discoloration, deterioration, damage or staining.

### 2) Adverse events

Corneal epithelial disorders such as corneal ulcers, corneal abscesses, corneal perforations, corneal infiltration, corneal erosions, keratitis, and corneal epithelial staining and corneal edema, corneal angiogenesis, conjunctivitis, subconjunctival hemorrhage, iriditis, wheat granuloma, mybome adenitis, chalazion, ptosis, regulatory eye strain, dry eye, decreased corneal endothelial cells.

## [ Symptoms and measures to be taken when using the lens. ]

- 1) Do not use of the lens if your eyes are uncomfortable.
- 2) If you any experiences of the following symptoms while using the lens, please try the followings. If symptoms persist, go to an eye doctor as soon as possible.

Symptoms	Measures
foreign body sensation, pain, itching, dullness, frequent eye discharge, redness	<ul style="list-style-type: none"> <li>- Check the lens and replace it with a new one if it is scratched or damaged.</li> <li>- Scrub and rinse the lens if there is any dirt.</li> <li>- If the time of application is too long, shorten the application time.</li> </ul>
instability of vision, difficulty viewing, eye fatigue	<ul style="list-style-type: none"> <li>- Check the left/right and front/back side of the lens and re-wearing.</li> <li>- If the time of application is too long, shorten the application time.</li> </ul>
dry feeling	<ul style="list-style-type: none"> <li>- Blink your eyes a few times and let out a lot of tears.</li> <li>- Dropping eye drops for soft contact lenses.</li> </ul>

## 3. Application to pregnant women, nursing women, lactating women, children etc.

- 1) If you need to take medication or eye drops, or if you are pregnant or immediately after childbirth, consult your eye doctor as this may affect your lens wear.
- 2) When children are allowed to use lenses, they should be supervised by guardian.

## 4. Other notes

- 1) As preparation when if you were loss the lens or stop wearing it, carry a spare lens and use it with your glasses.
- 2) Be wear lens before the makeup and remove the lens before removing the makeup.
- 3) Make sure that cosmetics do not get on the lens.
- 4) When using eye drops, we recommend using artificial tears designated to soft contact lenses of non-including preservatives.
- 5) Remove the lens when swimming.
- 6) The tinted part of the lens may reduce the amount of light and make it difficult to see. It may also cause a sense of discomfort in the peripheral areas of the visual field. These may decrease as you get used to wearing the lens, but if they persist, consult an eye doctor.
- 7) Depending on the use of the lens, it may be difficult to see in the dark. Be especially careful when driving a car or operating a machine in a dark place.
- 8) During strenuous sports or other activities, the lens may shift and obstruct your vision.
- 9) If you have an allergic disease, consult an ophthalmologist as there is an increased risk of an adverse event occurring.
- 10) If you are using it overseas, please bring the necessary care products with you.

- 11) If you are staying abroad for an extended period of time, you should have regular checkups at a local medical institution.
- 12) However, in the unlikely event that there is a defect in an unused product, please check the date and time of purchase, check the place of purchase, and keep the original lens, box, and lens container and contact the place of purchase.

### Storage method and Expiry, etc.

1. Storage method : Avoid direct sunlight, high temperatures, high humidity, and freezing and store at room temperature.
2. Expiry (EXP.) : The expiration date of the lens is indicated on the outer box of the lens and on the container itself. Do not use lenses that are past their expiration date. For example, YYYY/MM expires on the last day of month MM, year YYYY.

### Maintenance and Inspection

1. How to disinfect : Scientific Disinfection.
2. Lens care : Please refer to above “Lens care”.

### Symbol Reference Guide on Packaging

<b>P</b>	Lens Power	<b>EXP</b>	Expiry
<b>DIA</b>	Diameter		Caution (See product instruction)
<b>BC</b>	Base Curve		Sterilized using steam or dry heat
	Manufacturer		

### Warnings

1. Wearing contact lenses may cause corneal epithelial disorders such as corneal ulcers, keratitis, corneal infiltration & corneal erosions, corneal edema, conjunctivitis, iritis, corneal angiogenesis and so on, and may hasten the decrease of corneal endothelial cells. Some eye disorders caused by lens wear can lead to blindness if left untreated.
2. In order to avoid the eye damage described above, the following should be observed when using the lenses.
  - 1) **Ensure proper wearing time.**  
There is an individual difference in the time to wear the lens. Observe the wearing times indicated by your eye doctor.
  - 2) **Observe the period of use.**  
This lens has a fixed duration of use. Be sure to replace your lenses with new ones periodically, as directed by your eye doctor, without exceeding the duration of use.

- 3) **Observe the way of handling and use it correctly.**  
Improper handling of the lens and care products can lead to eye damage. Keep the lens and care products (especially the lens case) clean and use them in the correct way.
- 4) **Get a regular eye checkup.**  
Even if there are no symptoms and you are using it well, your eyes and lenses may be damaged or your eye damage may have progressed. Even if there are no abnormalities, be sure to have regular examinations as directed by your eye doctor.
- 5) **If you feel any abnormality, receive eye care immediately.**  
Before wearing the lenses, check to see if there are any smears or redness in the eyes, and after wearing the lenses, check to see if there is any foreign body sensation, etc. If you feel any abnormality, you should consult an eye doctor.
- 6) **Never use a lens that is damaged or otherwise defective.**  
Be sure to check the lens for damage or other defects before use. If any subjective symptoms such as lens damage occur while wearing the lens and it is not improved, and If you feel any abnormality, you should consult an eye doctor.

## Prohibition

1. **Applicants : The following people should not use it.**
  - ✓ Acute and subacute inflammation of the anterior segment of the eye.
  - ✓ Eye infection.
  - ✓ Uveitis
  - ✓ The corneal image is low.
  - ✓ Dry eye and lacrimal organ disease to the extent that becomes a problem for lens wear.
  - ✓ Eyelid abnormalities.
  - ✓ Allergic conditions to a degree that affects wear lens.
  - ✓ People who live in a constantly dry environment.
  - ✓ People who live in an environment where dust, chemicals etc. can be easily gotten into eyes.
  - ✓ People who cannot follow instructions from their eye doctor.
  - ✓ People who cannot use the lens properly.
  - ✓ People who cannot take a regular checkup.
  - ✓ People who are unable to maintain the hygiene required for lens wear.
2. **Use of Lens**
  - ✓ Do not give your lenses to others or use someone else's lenses.
3. **Concomitant Medical Device**
  - ✓ Remove the lens when you have an MRI.

- **Legal Manufacturer :** ANW CO., LTD.  
1-17-1, Toranomom, Minato-ku, Tokyo, 105-6414 JAPAN  
Tel : 81-3-6407-1669
- **Manufacturing Site :** YUNG SHENG OPTICAL CO., LTD.  
No.8 Keya 2nd Rd., Daya District, Taichung City,  
42881 TAIWAN
- **Malaysia Authorized Representative :** ARQon Medtech Sdn. Bhd.  
No. 42-02, Jalan Adda 7, Taman Adda Heights,  
81100 Johor Bahru, Johor, Malaysia  
info@arqon.com

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- **Pengilang Sah:** ANW CO., LTD.
- **Tapak Pengilangan:** YUNG SHENG OPTICAL CO., LTD.